

Possible means of application



Take off the screw cap and pour LAVANID® solution on to a sterile gauze to clean the wound.



Draw up LAVANID® solution under aseptic conditions using a spike and syringe and then use to clean the wound.



Squeeze LAVANID® V+ wound gel out of the tube on to a sterile spatula or sterile gauze and apply to the wound.



Apply LAVANID® wound gel directly from the sterile prefilled 10 ml syringe.

Product range



Pack sizes					
LAVANID® 1	PZN	REF	LAVANID® 2	PZN	REF
20 x 125 ml	05901297	014124	20 x 125 ml	05901274	014224
10 x 250 ml	02178452	014125	10 x 250 ml	02178564	014225
6 x 1000 ml	02178498	014127	6 x 1000 ml	02178587	014227

Pack sizes		
LAVANID® wound gel	PZN	REF
1 x 100 g	08837192	015274

Pack sizes		
LAVANID® wound gel V+	PZN	REF
1 x 100 g	10329497	015774
1 x 40 g	10813543	015773

Pack sizes		
LAVANID® wound gel prefilled syringe	PZN	REF
2 x 10 g	06122661	015272
12 x 10 g	02078785	015271

Contents and case reports  
Sabine Laudes  
Wundmanagement Klinikum  
Frankfurt-Höchst GmbH



SERAG-WIESSNER GmbH & Co. KG  
Zum Kugelfang 8 - 12  
95119 Naila/Germany

+ 49 9282 937-0  
+ 49 9282 937-9369

Export Department:  
+ 49 9282 937-230  
+ 49 9282 937-9785

info@serag-wiessner.de  
www.serag-wiessner.de

Examples of use in state-of-the-art

WOUND MANAGEMENT

Solutions



LAVANID® 1/2  
irrigation solutions

LAVANID®  
wound gel

LAVANID®  
wound gel V+

Stand 2016 - 03

Art.Nr. 851191





## Examples of chronic wounds

**Patient:** 42-year-old man  
**Comorbidity:** arterial hypertension  
**Description of wound:** wound over the right tibia following a road traffic accident, mesh grafting failed; length: 12 cm, width: 6.4 cm, slightly reddened wound margins, moderate exudate, not painful, wound present for about six months



<b>Day 0:</b> <b>Treatment:</b> wound cleaned with LAVANID® 2 solution, autolytic débridement with LAVANID® wound gel, covered with film dressing	<b>Day 67:</b> <b>Treatment:</b> wound cleaned with LAVANID® 2 solution, LAVANID® wound gel, covered with polyurethane foam	<b>Day 175:</b> <b>Treatment:</b> wound cleaned with LAVANID® 1 solution, LAVANID® wound gel dressing
--	--	--

**Patient:** 64-year-old woman  
**Comorbidity:** arterial hypertension, COPD  
**Description of wound:** wound on left lower leg, length: 12.3 cm, width: 8.7 cm, slightly reddened wound margins, moderate exudate, moderately painful, wound present for about three months



<b>Day 0:</b> <b>Treatment:</b> wound cleaned with LAVANID® 2 solution and LAVANID® wound gel, covered with polyurethane foam	<b>Day 18:</b> <b>Treatment:</b> wound cleaned with LAVANID® 2 solution, covered with contact layer dressing, sterile gauze dressing and fixation	<b>Day 63:</b> <b>Treatment:</b> wound cleaned with LAVANID® 1 solution, no dressing
--	--	---

## Examples of infected wounds

**Patient:** 75-year-old man  
**Comorbidity:** arterial hypertension  
**Description of wound:** venous leg ulcer over the right medial malleolus; length: 5.4 cm, width: 5.2 cm, slightly reddened wound margins, moderate exudate, not painful, wound present for about two years



<b>Day 0:</b> <b>Treatment:</b> wound cleaned with LAVANID® 2 solution, moist LAVANID® 2 wrap for three days, LAVANID® wound gel V+, covered with polyurethane foam, sterile gauze dressing and fixation	<b>Day 34:</b> <b>Treatment:</b> wound cleaned with LAVANID® 2 solution, LAVANID® wound gel V+, wound edge protection, wound covered with polyurethane foam, sterile gauze dressing and fixation	<b>Day 58:</b> <b>Treatment:</b> wound cleaned with LAVANID® 2 solution, LAVANID® wound gel, wound edge protection, wound covered with polyurethane foam, sterile gauze dressing and fixation
---	---	--

**Patient:** 63-year-old woman  
**Comorbidity:** Clostridium difficile infection, arterial hypertension, bipolar disorder  
**Description of wound:** large wound over the buttocks. The patient had been lying in her apartment in her own faeces for four days, resulting in a grade 3 pressure ulcer.



<b>Day 0:</b> <b>Treatment:</b> wound cleaned with LAVANID® 2 solution, moist LAVANID® 2 wrap and LAVANID® wound gel applied from prefilled syringe, covered with sterile gauze dressing	<b>Day 91:</b> <b>Treatment:</b> wound cleaned with LAVANID® 2 solution and LAVANID® wound gel applied from prefilled syringe, covered with polyurethane foam	<b>Day 124:</b> <b>Treatment:</b> wound cleaned with LAVANID® 1 solution and LAVANID® wound gel applied from prefilled syringe, covered with polyurethane foam
---	--	---

## Examples of burns

**Patient:** 83-year-old woman  
**Comorbidity:** arterial hypertension, COPD, atrial fibrillation, oral anticoagulation with Marcumar (phenprocoumon)  
**Description of wound:** second degree burn on the left thigh; length: 28 cm, width: 26 cm, moderate exudate, wound warm and very painful



<b>Day 0:</b> <b>Treatment:</b> wound cleaned with LAVANID® 1 solution, moist LAVANID® wrap for three to four days. Then LAVANID® wound gel V+, wound covered with contact layer dressing, sterile gauze dressing	<b>Day 6:</b> <b>Treatment:</b> wound cleaned with LAVANID® 1 solution and LAVANID® wound gel V+, wound covered with contact layer dressing, sterile gauze dressing and fixation	<b>Day 14:</b> <b>Treatment:</b> wound cleaned with LAVANID® 1 solution, good skin care, no dressing
--	---	---

**Patient:** 67-year-old man  
**Comorbidity:** syncope with cardiac arrhythmia  
**Description of wound:** large second degree burn on the right side of the abdomen; length: 19.8 cm, width: 29 cm, moderate exudate, wound hot and very painful



<b>Day 0:</b> <b>Treatment:</b> wound cleaned with LAVANID® 1 solution, moist LAVANID® wrap for three days, then LAVANID® wound gel V+, wound covered with contact layer dressing, sterile gauze dressing and fixation	<b>Day 5:</b> <b>Treatment:</b> wound cleaned with LAVANID® 1 solution, LAVANID® wound gel V+, wound covered with contact layer dressing, sterile gauze dressing and fixation	<b>Day 12:</b> <b>Treatment:</b> wound cleaned with LAVANID® 1 solution, good skin care, no dressing.
---	--	--